ART. 4CS.446.xxxx.xx

Users manual Rev.4 2020-09

MPC.IMB.ISTRUZ.117

SEGGIOLINO DA AGGANCIARE / SUSPEND SEAT / SIÈGE À SUSPENDRE / DUSCHSITZ EINHÄNGBAR

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EN INSTRUCTIONS

Read before installation, use and maintenance. This document is an integral part of the product: please keep it. All sizes are in mm. If you cannot read this document, it is possible to download this documentation on our website www.pba.it.

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EN
This product is comply with the directive 93/42/EEC for medical device.

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EN INTENDED USE

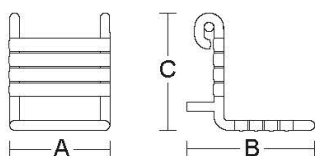
The suspended shower seats are a valid aid for elderly or disabled people, and are intended to be used by an adult. They are fixed using PBA safety supports in shower areas (please refer to fitting instructions): distance of the auxiliary supports does not have to exceed 800 mm. The shower seats offer the possibility for people with or without any physical handicap to sit whilst showering.

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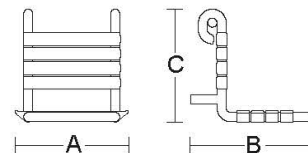
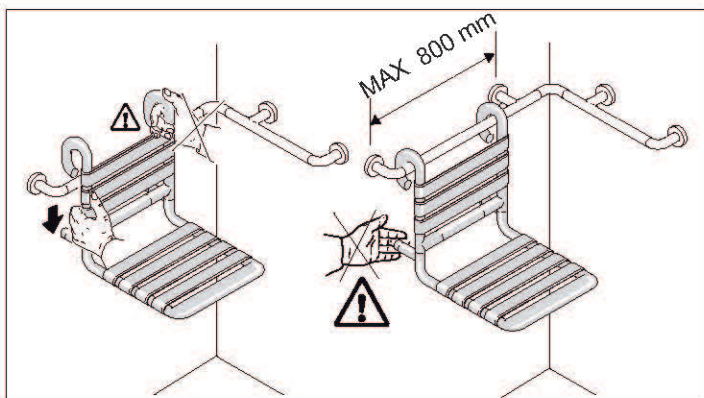
EN - APPLICATIONS AREA

Hospitals, rehabilitation centres, nursing home for elderly, public buildings, private bathrooms.

Movimentare e utilizzare l'ausilio come da immagine: attenzione schiacciamento dita. Move and use the safety support as shown in the picture: attention to squeezing fingers. Déplacer et utiliser l'aide comme indiqué sur l'image: attention aux doigts écrasés. Bewegen und verwenden Sie Stützklappgriffe wie im Bild angegeben: Achtung vorm Quetschen der Finger.

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EN SAFETY PRECAUTIONS AND INDICATIONS FOR MAINTENANCE

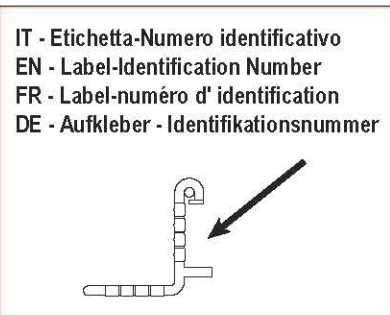
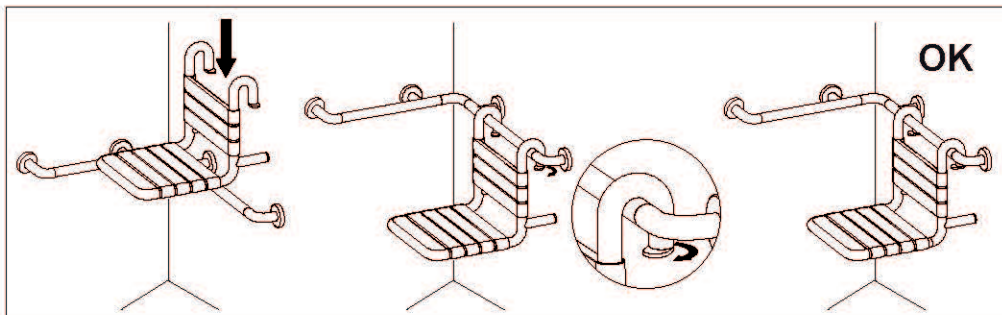
WARNING: All disabled accessories must be fixed in accordance with the table of fixing materials. pba S.p.A. is not liable for any damage/loss of functionality of the product caused by not observing the recommendations set out in our instruction manual. Under NO circumstances must the pba S.p.A. disabled and bathroom accessories be altered, tampered with or modified. Non-observance of these instructions makes product guarantees null and void.

⚠ Before installing the item, write down the serial number on the provided form.

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EN

Do not use the shower seats improperly or for any different use other than those indicated in the instruction manual. Showerseats must not be installed inside saunas or in extremely hot areas (a temperature of the support's surface exceeding 41°C could cause skin damages). In case of malfunctioning of our showerseats, please contact the technical department at pba S.p.A. (Tel. +39 (0)424 5451) or pba Deutschland GmbH (+49 2841 998900). It is recommended to check the correct functioning of the shower seat periodically (once a month) and that the fixation at wall of the support is not damaged or loose. Overloading the seat may cause permanent damage to the shower seat and to the fixing system. In this case, please contact pba S.p.A. (Tel. +39 (0)424 5451) or pba Deutschland GmbH (+49 2841 998900). Failure to regard these precautions may cause serious damage or accidents. **WARNING:** Please do not leave your child unattended. There is a risk of injury. The supervision of an adult is required.

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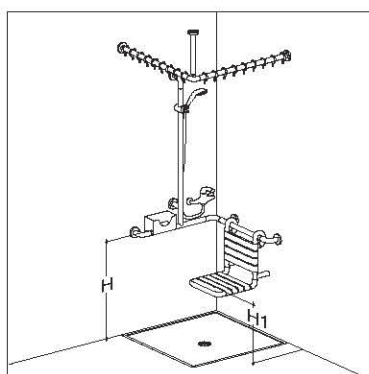
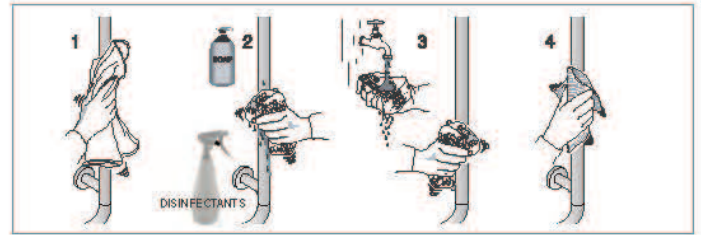
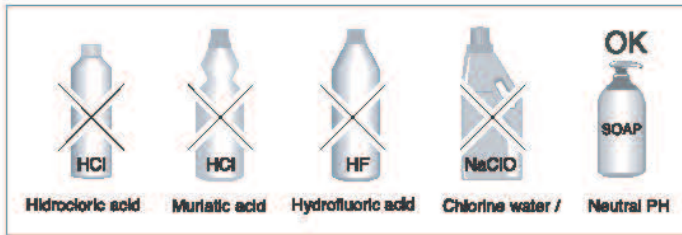
EN

INDICATIONS FOR CLEANING

The products only require simple cleaning and maintenance.

To clean pba products periodically use a wet cloth. As an alternative use neutral detergents or disinfectants diluted in a solution having neutral PH. After washing rinse the products with cool water and let them dry. Always select the correct soaps/disinfectants in accordance with the manufacturer's instructions (ensuring that they are compatible with the different surfaces and correct concentration). It is recommended to test the detergent/disinfectant on a small hidden area of the product before use. Wash the part in the direction of the satin finish to avoid scratching the surfaces. The use of detergents containing chlorine, acids and alkali, organic solvents and abrasive substances is not recommended. Do not use any abrasive objects (like brushes) for the cleaning.

Furthermore, avoid using abrasive sponges or steel wool, as these will contaminate the surfaces of the material. Finally, stubborn deposits on the surfaces of the part must be removed using a wood spatula (not a steel one), to avoid scratching and contamination. Pba S.p.A. declines all responsibility for damages caused by use of unsuitable cleaning methods listed above.

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*Standard PBA

Estimated pit corrosion penetration time - time to penetrate 1 mm (years) by steel type

Material - Stainless	Marine	Semi_industrial	Rural
AISI 430 (1.4016)	N/A	85	250
AISI 304 (1.4301)	145	135	770
AISI 316 (1.4401)	260	525	1200

Source: The British Stainless Steel Association (BSSA)

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EN The AISI 316L stainless steel (1) steel used by pba, meets the requirements of UNI EN 1670, the ultimate quality standard for ensuring resistance to corrosion. Pba has been awarded a certificate for successfully salt spray testing the stainless steel in accordance with UNI ISO 9227.
(1) X2CrNiMo 17-12-2 as per EN 10088-3

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EN INFORMATION FOR THE DISPOSAL OF WASTE PRODUCTS

At the end of its life, the product must be disposed according the local waste regulation. The article is made of AISI 316L stainless steel and recyclable polyamide.

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EN Under the intended conditions of use and in accordance with the guidelines for maintenance and cleaning, the expected life time of the product is of 5 years.

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WALL RESISTENCE

The wall must have sufficient strength to ensure the application of 250 kg at each fixing point.

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MAINTENANCE

- 1) Verify the correct anchorage of the 4CS.440 to the wall
- 2) In case of breakage of any component or in case of anomalies, contact immediately the manufacturer and do not use the support.
- 3) It is recommended to carry out maintenance checks every 1 year. Record the checks carried out in the appropriate attached form.

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pba S.p.A. - I-36056 Tezze sul Brenta (VI) - Via Enrico Fermi, 1
pba Deutschland GmbH - Alter Graben 47-49 - D-47918 Tönisvorst
pba USA, Inc. 68 34th - Street Suite C405 - Unit 20 - Industry City - Brooklyn NYC - NY 11232